

Lift Remote Programming



REMOTE TYPE #1

1. Place battery on lift with switch in the off position. (For additional safety, during programming, ensure that only the lift being programmed has a battery installed.)
2. Press and hold unlock button for 3-4 seconds *and release*.
3. Press and hold the UP button on the remote while switching the battery to the ON position.
4. The lift should begin to move UP. If it does not, not repeat steps 1-3 until the lift moves UP when the battery is switched ON.



REMOTE TYPE #2

1. Place battery on lift with switch in the off position. (For additional safety, during programming, ensure that only the lift being programmed has a battery installed.)
2. Press and hold the UP button on the remote while switching the battery to the ON position.
3. Continue holding the UP button for 10-15 seconds.
4. The lift should begin to move UP. If it does not, not repeat steps 1-3 until the lift moves UP when the battery is switched ON.

To remove a programmed remote:

1. Switch the battery on the lift into the OFF position. (For additional safety, during programming, ensure that only the lift being programmed has a battery installed.)
2. Press and hold the UP button on the remote while switching the battery to the ON position.
3. Continue holding the UP button for 10-15 seconds.
4. The lift should begin to move UP. If it does not, repeat steps 1-3 until the lift moves UP when the battery is switched ON.
5. Repeat steps 1-4 five times.

If you have any difficulties, please contact Spectrum Customer Service at 406-532-6321.

2/2016